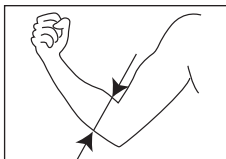
# GIBAUD

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# GIBAUD

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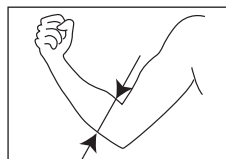
Measurement exact circumference

## ELBOWGIB®

**If you have any doubts concerning the application and use of this medical device, do not hesitate to consult your doctor and/or your fitter, pharmacist - orthotist - orthopaedist.**

### FEATURES AND BENEFITS:

#### Efficacy:

- The counterforce effect provided by the two cushions reduces traction on the tendons and thus inflammation;
- The undulated design of the cushions filters the vibrations transmitted to the tendons.

#### Comfort:

- The patented double-curve design of the strap eliminates tourniquet effect; the strap can be worn at rest;
- Counterforce bracing is provided by the large cushions situated on opposite sides of the forearm.

### Design:

- Ergonomic design eliminates the tourniquet effect caused by muscle swelling during exertion;
- This product was designed for sporting activities.

### INDICATIONS:

- Lateral epicondylitis (tennis elbow);
- Medial epicondylitis (golfer's elbow);
- Resumption of activities;
- Preventively at work, for everyday and sporting activities.

### USE:

- Pull the strap onto the forearm;
- The cushions should be placed over the painful tendons (just below the elbow);
- Tighten using the flap to obtain firm but comfortable support.

### CONTRAINDICATIONS AND RESTRICTIONS ON USE:

- Any injury requiring orthopaedic treatment with a plaster cast, resin cast or surgery;
- The strap must not be in direct contact with a skin lesion.

### CARE AND MAINTENANCE:

- Store the strap away from sources of excessive heat;
- Hand wash in warm soapy water (maximum 40°C). Dry flat, away from sources of excessive heat (radiator, tumble dryer).

### SIZES:

To calculate the size, measure the circumference at the upper third of the forearm and refer to the table below:

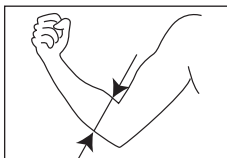
Circumference at the upper third of the forearm in inches	8.7-10.4	10.4-12.8	12.8-15
Size	1	2	3

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