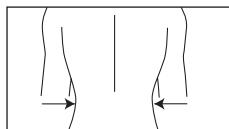
GIBAUD

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GIBAUD

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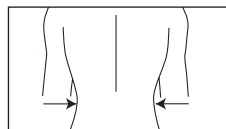
Measurement exact circumference
in inches

LOMBOGIB® ARTHROSE

If you have any doubts regarding the
applications and use of this medical
device, do not hesitate to ask your
doctor and / or fitter, pharmacist,
orthotist, or orthopaedic consultant
for advice.

INDICATIONS:

- Hyperalgesic patient.
- Osteoarthritis of the lumbar spine (multilevel degenerative disc disease).
- Chronic pain syndrome involving the lumbar spine (lower back pain, sciatica, post-traumatic, post-surgical).

FEATURES AND BENEFITS:

Efficacy:

- Stabilisation of the painful segment by the lumbosacral reinforcement, supported by the iliac crests.
- Unloading of the painful segment through an patented innovative compression system.
- Anatomical conformity thanks to the compression system that prevents excessive constriction (creation of an hourglass shape).

Comfort:

- Easy positioning for persons unable to apply significant muscular force (arthritis; strong pain; age) thanks to its patented system.
- Elimination of the sensation of abdominal constriction by compression specifically targeting the lumbar area (lumbosacral reinforcement).
- Anatomically shaped for improved comfort in the sitting position.
- Ventilated perforated fabric.

Design:

- Intuitive and ergonomic fastening system.

PRECAUTIONS FOR USE:

- Progressive withdrawal, possibly accompanied by functional rehabilitation.
- Appropriate protection of skin lesions.

RECOMMENDATIONS:

In addition to taking measurements, it is essential to try on the belt to adapt it correctly and comfortably, and to ensure that the positioning is correctly understood.

POSITIONING:

Before application: open the belt, lay it flat, detach the two side loops and separate the two parts of the belt.

Position the belt by placing the strap with the Gibaud logo on the right and aligning the opening of the back panel with the spinal column.

The lower edge of the back panel should be level with the top of the buttock crease.

Close the belt without tension.

Locate the side straps, place the thumbs in the loops with the hands pointing to the front; pull forwards on the straps to obtain firm but comfortable compression.

Attach the side straps to the front of the belt.

The tension in the belt can be adjusted in accordance with the pain level and activities.

To remove the belt:

- Detach the side straps and let them hang free;
- Open and remove the belt.

CARE AND MAINTENANCE:

- Keep the belt away from sources of excessive heat.
- Hand wash.
- Dry flat, away from excessive heat (no radiators, no dryers).

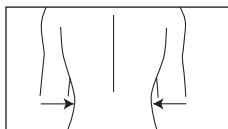
Size	T1	T2	T3	T4	T5
Waist circumference in inches	25.6-33.5 inches	31.5-37.4 inches	35.4-41.3 inches	39.3-45.3 inches	43.3-49.2 inches

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