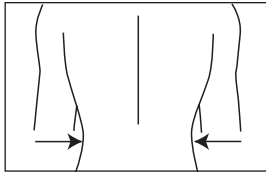
GIBAUD

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[FR]

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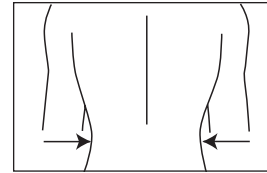
GIBAUD

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[EN]

Measurement exact circumference in inches

- Ergonomic finger pockets for easier fitting.

INDICATIONS:

- Acute low back pain.
- Sciatica (worn continuously).
- Chronic low back pain (worn intermittently during routine day-to-day activities).

FITTING:

- In addition to measuring the waist, the belt should be tried on to ensure a good fit and to understand how to put it on properly.
- The belt is worn under the clothes or over the undergarments.
- To put it on, centre the dorsal rib on either side of the spinal column.
- The Gibaud logo must be on the right.
- Tighten the belt based on the intensity of the pain and type of activity.

CONTRAINDICATIONS AND LIMITATIONS OF USE:

- Pelvic organ prolapse.
- Pregnancy.
- Extremely thin person.
- Respiratory difficulties.

PRECAUTIONS FOR USE:

- Progressive withdrawal, possibly accompanied by functional rehabilitation.
- Appropriate protection of skin lesions.

RECOMMENDATIONS:

In addition to measuring the waist, the belt should be tried on to ensure a good fit and to understand how to put it on properly.

SIZE:

Measure the waist in centimetres and refer to the size table below. The measurement should be made two fingers above the iliac crest. If the hip measurement is 10cm greater than the waist measurement (at the navel), use the hip measurement for the measuring process.

CARE AND MAINTENANCE:

- Keep the belt away from direct heat sources.
- Hand wash or machine wash at 30°C. Dry flat, away from direct heat sources (radiators, dryers).

LOMBOGIB® UNDERWEAR

If you have any doubts about how to apply or use this medical device, do not hesitate to consult your doctor and/or your fitter, pharmacist - orthotist - orthopaedist.

6532/6655

PROPERTIES:

- Patented, single-layer Aeralis fabric that is breathable, soft and can be comfortably worn directly against the skin (better compliance).
- V-shaped ribs to focus the effect on the high-risk area (lumbosacral joint).

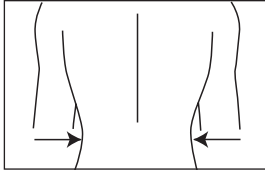
Height (inches) : 10.24						
Waist size in inches	27.6-31.5	31.5-35.4	35.4-39.4	39.4-43.3	43.3-49.2	49.2-55.1
Size	0	1	2	3	4	5

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