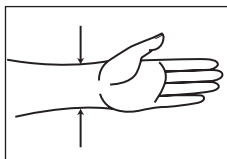
GIBAUD

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GIBAUD

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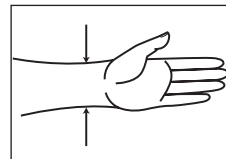
Measurement exact circumference

- Palm free to preserve prehension and interphalangeal flexion.

Confort:

- Fleece lining providing warmth; breathable material releases moisture.
- Light, ventilated, breathable 3D fabric, contoured to hand anatomy.

Design:

- Easy application with its circular design and unique hook and loop flap.

INDICATIONS:

Basal thumb arthritis:

- Subacute phase: intermittent wear depending on pain.
- Chronic phase: Day wear: joint load reduction, and support in everyday and work tasks.

USE:

- Pull the brace on the hand like a glove.
- Tighten using the flap to obtain firm but comfortable support.
- Tighten the thumb strap to obtain firm but comfortable support.

CONTRAINDICATIONS AND RESTRICTIONS ON USE:

- Any injury requiring orthopaedic treatment with a plaster cast, resin cast or surgery.
- The brace must not be in direct contact with a skin lesion.

CARE AND MAINTENANCE:

- Store the brace away from sources of excessive heat.
- Hand wash in warm soapy water (maximum 40°C). Dry flat, away from sources of excessive heat (do not use a radiator or tumble dryer).

MANUGIB® BASAL THUMB ARTHRITIS FUNCTION BRACE

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

FEATURES AND BENEFITS:

Efficacy:

- Stabilization of the base of the thumb and wrist (encourages pronation and supination).

SIZES:


To calculate the size, measure wrist circumference and refer to the table below.
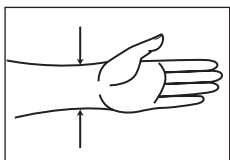
| Wrist circumference in inches | 5.1 - 7.1 | 7.5 - 8.7 |
|-------------------------------|-------------------------|-------------------------|
| Size | 1D - Right 1G - Left | 2D - Right 2G - Left |

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