GIBAUD

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## TAILLE

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**If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice**

## INDICATIONS

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre or post-operative.
- To support a cast or a splint.

## PERFORMANCE

The device shall support the arm.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Competent Authority of the Member

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

- 1 - First slip the forearm into the fixed loop close to the elbow.
- 2 - Pass the strap up the back and over the opposite shoulder.
- 3 - Position the adjustable strap at the wrist.
- 4 - Adjust the angle of the forearm (90°) using the sliding buckle.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U