GIBAUD

# CEINTURE CHALEUR THERMAL BELT

4290, 4291, 4292, 4293, 4294, 4295, 4296, 4297

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**EN** | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

## INDICATIONS

For conditions that may benefit from the adiathermic properties of the wool, such as:

- Low back pain.

## PERFORMANCE

The thermal belt provides low back pain relief.

Thermal insulation and breathability.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

This product contains Naturel Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up.

The belt must not be used in direct contact with a damaged skin.

The belt is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the belt, e.g. pain or appearance of local signs, remove the belt and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent

authority of the Member State in which the user and/or patient is established.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

When fitting, put the belt on by the legs, pull it up until the bottom of the belt is positioned at the level of the intergluteal cleft, covering the painful area.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the belt away from sources of excessive heat.

Hand wash in warm soapy water (maximum 30°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

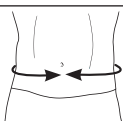
## MEASUREMENT AND SIZE CHART

The device is available in three heights:

- 20 cm (4290/4291),
- 25 cm (4292/4293/4294/4295/4296),
- 30 cm (4297).

The size is to choose in accordance with the waist circumference, measured 2 cm above the iliac crests (in cm)

Height (cm)		Waist circumference (cm)				
		S	M	L	XL	XXL
20		61-75	75-90	90-105	105-120	
25		61-75	75-90	90-105	105-120	
30			75-90	90-105	105-120	120-135

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GIBAUD S.A.S.U