## CHAUSSETTES de COMPRESSION SPORTIVE

GIBAUD

PR13004  
V1\_202004

## CHAUSSETTES de COMPRESSION SPORTIVE

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The technical product line calls on GIBAUD's knowledge and specialization in Medical Compression to serve athletes.

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### PRODUCT CARE AND DISPOSAL

Delicately hand wash, rinse thoroughly and gently wring out. Machine wash on a delicate cycle up to 40°C. Do not use fabric softener or bleach. Do not dry clean or iron. Dry flat, away from sources of heat. Do not tumble dry. Dispose of the device and its packaging in accordance with local or national environmental regulations.

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### PUTTING ON THE DEVICE

Correctly position the device over the toes, then the heel. Make sure the material is stretched evenly over the leg. The Gibaud Sport branding should be centred at the front and the support zone should cover the calf.

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### INDICATIONS:

Devices are intended for persons who may benefit from compression therapy during sport activities and/or recovery phase.

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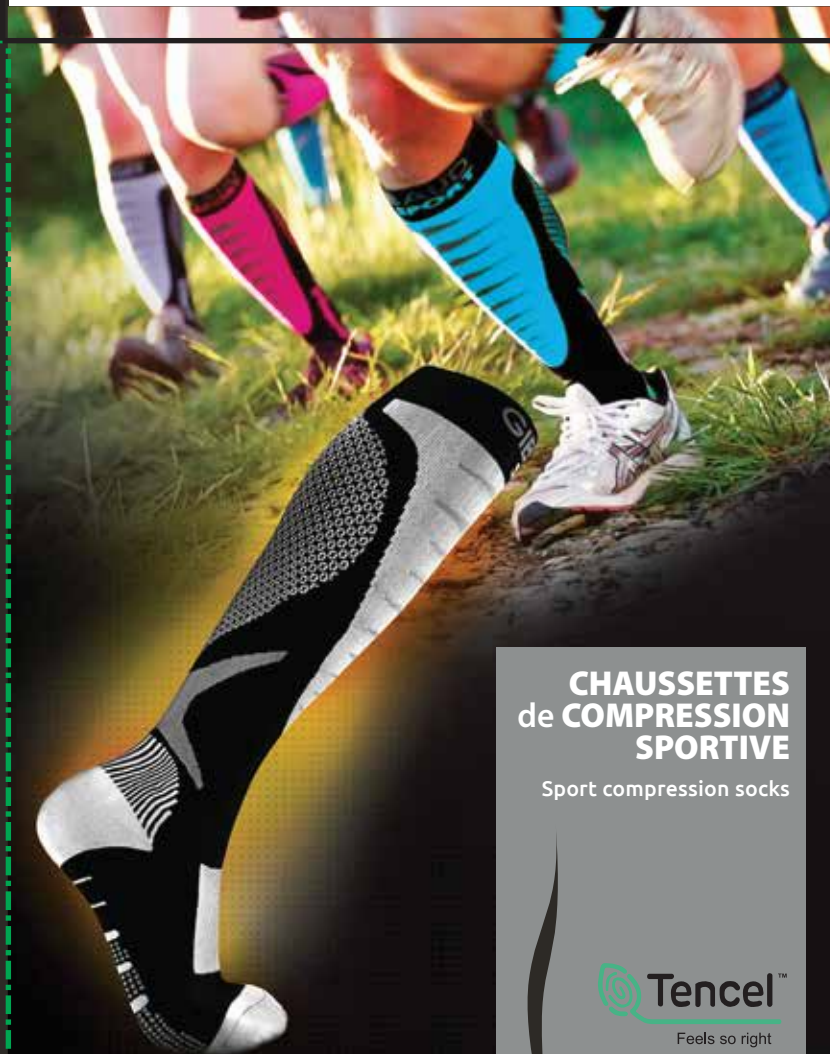
### CONTRAINDICATIONS:

Must not be used by individuals for whom compression is contraindicated. • Obliterative arteriopathy of the lower limbs with a systolic pressure index (SPI) < 0.6. • Phlegmasia cerulea dolens (fulminant condition of venous thrombosis). • Septic thrombosis/phlebitis. • Peripheral neuropathy and sensitivity disorders. • Compensated congestive heart failure. • Diabetic microangiopathy.

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### PRÉCAUTIONS:

If any problems occur with the application of the device, contact your healthcare professional. In the event of a complaint, please return the device in its packaging.

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### WARNINGS:

The device must not be used in direct contact with a damaged skin. If any problems occur with the use of the device, e.g. pain or appearance of local signs, remove the device and contact your healthcare professional. The device is a single patient use device. Do not use for multiple patients. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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