ELBOWGIB®

6478

- FR | Notice d'utilisation
- EN | Instructions for Use
- IT | Istruzioni per l'uso
- DE | Gebrauchsanweisung

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATION

- Epicondylitis.

PERFORMANCE

The device provides pain relief for lateral epicondylitis.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

When adjusting the strap, tighten it so that it is firm but comfortable.

The orthosis must not be used in direct contact with a damaged skin.

Do not wear for extended periods without medical follow-up.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

The device consists of a semi-rigid bracelet with patented double curvature, two viscoelastic cushions and a tightening strap with a hook-and-loop closure system.

SET UP AND OPERATION

The first trial should be carried out by a qualified expert to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user, a family member or a friend.

1) Pull the strap onto the forearm; the cushions should be placed on the tendons below the painful area.

2) Tighten to obtain firm but comfortable support.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store the brace away from sources of extreme heat (including radiators and tumble dryers).

Hand wash in warm soapy water (maximum 40 °C), rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

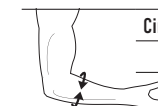
The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the circumference of the forearm below the elbow (in cm)

Circumference (cm)	22-26	27-32	33-38
Size	1	2	3

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