GIBAUD

GENUGIB® ARTHROSE

GENUGIB® ARTHRITIS

6558

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EN | Instructions for Use

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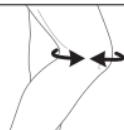
GENUGIB® ARTHRITIS

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATION

For knee conditions that may benefit from increased stability around the knee and shock absorption. This may include:

- Knee osteoarthritis.

PERFORMANCE

The knee brace provides increased stability around the knee.

The heel pad provides shock absorption.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires absolute immobilization or surgical procedure.

When adjusting the straps, tighten them so that it is firm but comfortable.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The orthosis is a single patient use device. Do not use for multiple patients.

This product contains Natural Rubber Latex which may cause allergic reactions.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Opens completely from the front.
Geared polycentric hinges.
Shock-absorbing heel pad.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During fitting, the applicator must shape, if necessary, the joints of the orthosis to adapt it to your form.

The brace is properly oriented when the GIBAUD logo is the right way up.

- 1) Open the brace up completely.
- 2) Place the two grey-coloured patellar half-rings on either side of the patella.
- 3) Close the lower part and then the upper part using the astrakhan and hook system.
- 4) Tighten the straps until the fit is snug but comfortable.
- 5) Place the heel pad in your shoes under your heels. The heel pad can be placed under the existing shoe sole for improved comfort.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep brace and heel pad away from direct heat sources.

After hinges have been removed from the orthosis, hand wash the orthosis and the heel pad in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from direct heat sources (ex: radiators), do not tumble dry; do not iron; do not dry clean.


The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the knee (in slightly bent position), measured at the middle of the patella (in cm).

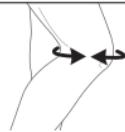	T1	T2	T3	T4	T5	T6
	31-34 cm	34-37 cm	37-40 cm	40-43 cm	43-46 cm	46-50 cm

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GIBAUD S.A.S.U