# GIBAUD

## GENUGIB® STAB

6554

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**EN** | Instructions for Use

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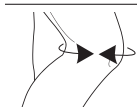
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

## INDICATIONS

For knee conditions that may benefit from increased stability around the knee, such as:

- Mild to moderate medial/lateral knee instabilities, sprains and strains.

## PERFORMANCE

The device shall provide increased stability around the knee.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

This product contains Natural Rubber Latex which may cause allergic reactions. The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

When adjusting the strap, tighten it so that it is firm but comfortable.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

Dual-centre hinge mechanism and stiff lateral struts increase stability around the knee.

Silicone patellar ring and groove for the patellar tendon.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

The brace is properly oriented when the GIBAUD logo is the right way up.

Loosen the straps.

Slip on the brace, position the patella in the center of patellar ring.

Tighten the straps to obtain firm but comfortable maintain.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

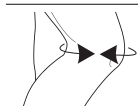
Hand wash in warm and soapy water (maximum of 40°C) after removing the hinges; rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

To determine the size, measure the circumference (cm) of a slightly bent knee and refer to the size table below.

1	2	3	4	5	6
31-34	34-37	37-40	40-43	43-46	46-50

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GIBAUD S.A.S.U