# MALLÉOGIB® 3D STRAP

## MALLÉOGIB® 3D STRAP ANKLE SLEEVE

6680

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EN | Instructions for Use

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SK | Návod na použitie

FR

## MALLÉOGIB® 3D STRAP

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FR

## MALLÉOGIB® 3D STRAP

**If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.**

### INDICATIONS

For ankle conditions that may benefit from compression and inversion restriction, such as:

- Mild and moderate (non-edematous) sprains
- Ankle injury rehabilitation
- Functional instability

### PERFORMANCE

- The device provides ankle support
- The device provides proprioception
- The device provides class 3 compression

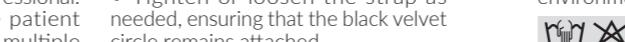
### CONTRAINDICATION:

Must not be used by individuals from whom compression is contraindicated

### WARNING • PRECAUTION OF USE

- When adjusting the strap, tighten it so that it is firm but comfortable
- The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.
- Do not wear for extended periods without medical follow-up.
- Now pass both ends of the strap around the heel. The top strap should cover the ankle pads, and the bottom strap the arch of the foot just before the heel.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations

EN

## MALLÉOGIB® 3D STRAP ANKLE SLEEVE

EN

## MALLÉOGIB® 3D STRAP ANKLE SLEEVE

EN

### FEATURES AND BENEFITS

- Patented external dynamic anti-varus strap prevents ankle rolling outwards.

### SET UP AND OPERATION

In addition to taking measurements, testing is required for a proper fit, and for the patient to learn the proper placement.

The ankle sleeve can be used on the left or right ankle.

- The ankle sleeve is put on by sliding it over the ankle like a sock. The Gibaud logo should be on the top of the ankle.

- Set the gripping rectangle on the cut-out corners with the foot icons printed on the black velvet rectangle on the back of the foot. When properly strapped up, the foot icon printed on the strap should match the foot on which the support is worn (right foot icon on right foot, left foot icon on left foot).

- Pull the strap in the direction indicated by the arrow on the foot print printed on the strap.

- Pass the strap under the foot, pulling it evenly to engage the dynamic external return.

- Pass the strap over the foot, top of the foot and place the black velvet circle over the rectangle where you started.

- Now pass both ends of the strap around the heel. The top strap should cover the ankle pads, and the bottom strap the arch of the foot just before the heel.

- The orthosis must not be used in direct contact with damaged skin.

- If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

- The orthosis is a single patient use device. Do not use for multiple patients.

- This product contains natural rubber Latex which may cause allergic reactions.

- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Competent Authority of the Member State is identified on this web site: [https://ec.europa.eu/growth/sectorsmedical-devices/contacts\\_en](https://ec.europa.eu/growth/sectorsmedical-devices/contacts_en)

	1	2	3	4	5
cm	18-20	20,5-23	23,5-26	26,5-29	29,5-32

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