GIBAUD

MANUGIB® RHIZARTHROSE IMMOBILISATION

MANUGIB® BASAL THUMB ARTHRITIS IMMobilIZATION

6684

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EN | Instructions for Use

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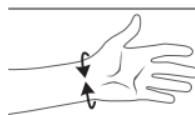
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATION

Thumb Carpometacarpal (CMC) Osteoarthritis.

PERFORMANCE

The brace provides stabilization of the thumb.

The brace shall provide functional improvements for CMC OA.

The brace provides pain relief for CMC OA.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

When adjusting the strap, tighten it so that it is firm but comfortable.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Adjustable commissure piece.

Conformable palmar and radial splint.

Palmar padd.

Single adaptation tab.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During fitting, the applicator must shape, if necessary, the splints of the orthosis to adapt them to your morphology.

1. Assess the damage by closing the first web contracture (thumb-index finger).
2. Depending on the damage, adjust the opening of the webbing spacer (aluminum cradle) to a position that relieves pain.
3. Open the wrist adjustment flap.
4. Put the brace on the hand like a glove.
5. Ensure that the spacer is correctly positioned at the base of the first web contracture (thumb- index finger).
6. Tighten the wrist adjustment flap to obtain a comfortable hold.

It may be necessary to adjust the inclination of the main spacer splint to prevent pressure on the trapeziometacarpal joint.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the orthosis away from sources of excessive heat.


Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean. The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the wrist, below the styloid (in cm).

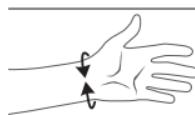1G - Left / 1D - Right	2G - Left / 2D - Right	3G - Left / 3D - Right
13,5-15,5	15,5-18	18-21

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GIBAUD S.A.S.U