ATTELLE 0° UNIVERSELLE UNIVERSAL 0° SPLINT

6910

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EN | Instructions for Use

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SL | Návod na použitie

FR

ATTELLE 0° UNIVERSELLE

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EN

UNIVERSAL 0° SPLINT

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For conditions that may benefit from knee immobilization, such as:

- Traumatic knee injuries.
- Emergency immobilization.
- Pre-/post-op knee immobilization.

PERFORMANCE

The device provides motion restriction of the knee joint or relative immobilization.

CONTRAINDICATIONS

None known.

WARNINGS • PRECAUTIONS OF USE

The splint should not be used when the patient's condition requires absolute immobilization or a surgical procedure. Do not wear for extended periods without medical follow-up.

The splint must not be used in direct contact with a damaged skin.

The splint is a single patient use device. Do not use for multiple patients.

When adjusting the straps, tighten them so that the hold is firm but comfortable.

If any problems occur while using the splint, e.g. pain or appearance of local signs, remove the device and contact your healthcare professional.

Store the splint away from excessive heat.

After removing the stays from the splint (see illustrative drawing), hand wash the splint in warm soapy water (maximum 40°C), rinse thoroughly; wring out gently; do not bleach; dry flat away from excessive heat (e.g. radiator); do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with local or national environmental regulations.

FEATURES OF THE DEVICE
3 flaps
Pairs of removable lateral metal stays (0° of flexion)
2 dorsal stays (1 for size T0)
1 pre-positioning strap
4 semicircular straps.

MEASUREMENT AND SIZE CHART

The size of the orthosis is to be chosen according to the length from the upper 1/3 of the thigh to the lower 1/3 of the calf of the leg to be immobilized (in cm).

	T0	T1	T2	T3	T4
	30	40	50	60	70

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