**GIBAUD**

# **BLOPAGE CLAVICULAIRE**

# **CLAVICLE BLOCKING PAD**

4222

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**EN** | Instructions for Use

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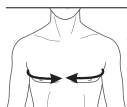
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice

## INDICATIONS

For conditions that may benefit from pulling the shoulders back, such as:

- Clavicle fractures and dislocations.
- Postural problems.

## PERFORMANCE

The device shall pull the shoulders back.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

- 1 - Position the back plate with the buckle between the shoulder blades.
- 2 - Detach the two clavicle pads from the straps.
- 3 - Pass the straps under the arms and insert them into the triangular buckle in the back plate.
- 4 - Reattach the clavicle pads to the straps.
- 5 - Adjust using the straps.

## CARE INSTRUCTION AND MAINTENANCE

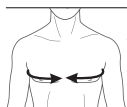
Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the chest (without the arms) in cm.

T0	T1	T2	T3	T4	T5	T6
45-55	55-65	65-75	75-85	85-100	100-115	115-135

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GIBAUD S.A.S.U