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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATION

For abdominal and low back conditions that may benefit from abdominal compression and gross motion restriction, such as:

- Abdominal strain or weakness of the abdominal area (post-partum, post-surgery) associated to: sciatica, lumbago / low back pain.

PERFORMANCE

The devices provide compression and/or support to the abdomen.

The devices reduce low back pain.

The devices provide lumbar spine gross motion restriction.

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS • PRECAUTIONS OF USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

The orthosis must not be used in direct contact with a damaged skin

This product contains Naturel Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Gradual withdrawal possibly combined with functional physiotherapy.

FEATURES OF THE DEVICE

The system consists of:

- A lengthwise elastic waistband with a standard height of 26 cm (6516/6516SM) or 21cm (6616/6616SM) at the front which covers the abdomen. It includes four rigid stays at the rear to support the lumbar region and three stays at the front.
- A front plate, comprising three rigid stays and a wide range of adjustment, which is fixed laterally by hook-and-loop fastening systems.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend. Wear over an undergarment or light clothing.

- 1- Position the bottom part of the belt at the top of the gluteal cleft
- 2- Center the dorsal ribs on either side of the spine. The Gibaud logo must be on the right side.
- 3- The belt must be closed by stretching the two sides symmetrically using the finger holes.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store the brace away from sources of extreme heat (including radiators and tumble dryers). Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the waist circumference, measured 2cm above the iliac crests (in cm).

	Waist circumference	88-100	>100
	Size	1 (CSB)	Refer to the made-to-measure service (CSA)
	Reference	6516/ 6616	6516SM/ 6616SM

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