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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For neck injuries that require motion restriction of the cervical spine such as :

- C-Spine precaution for trauma patients.
- Moderate neck injuries.
- Other conditions requiring gross immobilization of the mid-cervical spine.

PERFORMANCE

The device restricts range of motion of the cervical spine.

CONTRAINDICATION

The orthosis must not be used for traumatic injuries requiring absolute immobilization by plaster cast, resin cast or surgical procedure.

WARNING • PRECAUTION OF USE

The orthosis must not be used in direct contact with a damaged skin. The orthosis is a single patient use device. Do not use for multiple patients.

Do not wear for extended periods without medical follow-up.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs immediately contact your healthcare professional


The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Progressive withdrawal along with appropriate medical follow-up is necessary following prolonged wear for more than 10 days.

MEASUREMENT AND SIZE CHART

The size is determined by the user's neck circumference (in cm).

6596	6596 / 6606					
	00	0	1	2	3	4
29-32 cm	32-35 cm	32-38 cm	38-41 cm	41-43 cm	43-48 cm	

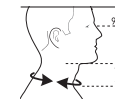FEATURES AND BENEFITS

The devices consist of two rigid plates that slide relatively to each other, allowing for height adjustment, and are held in place by three hook-and-loop fasteners.

The cervical collar C3 (6596) is without chin rest.

The C3 cervical collar (6606) with chin rest has additional support under the chin.

SET UP AND OPERATION

The first trial should be carried out by a qualified expert to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

- After selecting the size according to neck circumference, the height of two sliding

plates can be adjusted.

- The proper position is achieved when the chin rests comfortably on the upper pad,

the lower pad touches the sternum and the gaze is horizontal.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

30°C mild fine wash (Hand wash); rinse thoroughly, wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations

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