GIBAUD

LOMOGIB® EVOLUTION

6660/6661/6669/6670

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EN | Instructions for Use

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FR

LOMOGIB® EVOLUTION

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WARNING • PRECAUTION OF USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

The orthosis must not be used in direct contact with a damaged skin.

This product contains Natural Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Gradual withdrawal possibly combined with functional physiotherapy.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store the brace away from sources of extreme heat (including radiators and tumble dryers). Hand wash in warm soapy water (maximum 40 °C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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