GIBAUD

MANUGIB® TRAUMA POIGNET MANUGIB® WRIST TRAUMA

6508

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EN | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For conditions that may benefit from restricting motion of the wrist, such as:

- Mild to moderate sprains and strains.
- Carpal Tunnel syndrome.
- Tendonitis.
- Arthritis.
- Post cast healing injuries.
- Bursitis.
- Other Selected soft tissue injuries.

PERFORMANCE

The products shall restrict motion of the wrist.

The products shall allow full range of motion of the fingers.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

Patients should be instructed to inspect their hand, wrist, and product before and after each use.

The orthosis must not be used in direct contact with a damaged skin.

This product has been designed and tested as a single use product and is not intended for multiple use.

If any problems occur with the use of this product, immediately contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to

the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

- Wide, adjustable palmar stay.
- 3 tightening straps.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

Pull the brace onto the hand like a glove.

Tighten moderately using the straps; the palmar splint should be centred.

Successively adjust the straps to obtain firm and comfortable support.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the wrist circumference (in cm).

Size	0D - Right 0G - Left	1D - Right 1G - Left	2D - Right 2G - Left	3D - Right 3G - Left
Wrist circumference (cm)	11-13	14-16	17-19	20-22

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GIBAUD S.A.S.U