GIBAUD

EQUALIZER® COURTE

EQUALIZER® LOW TOP

6718

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EN | Instructions for Use

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EQUALIZER® LOW TOP

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

Foot and ankle conditions that may benefit from motion restriction, such as:

- Stable foot fractures.
- Soft tissue injuries.
- Post-operative, trauma or rehab use.

PERFORMANCE

The device provides immobilization/motion restriction.

The device has rocker bottom that provides natural and stable gait.

The device has shock absorbing sole.

CONTRAINDICATION

No absolute contraindications.

WARNINGS • PRECAUTIONS OF USE

Do not remove the device unless instructed by physician.

This product has been designed and tested based on single patient usage and is not recommended for multiple patient usage.

The orthosis must not be used in direct contact with a damaged skin.

If skin irritations of any kind develop, consult medical professional immediately.

Be sure the patient performs circulation checks. If a loss of circulation is felt, or if the walker feels too tight, release contact closure straps and adjust to a comfortable level. If discomfort continues, the patient should contact their medical professional immediately. When prescribed to patients with

diabetes, vascular deficiencies and neuropathies extra caution and attention to patient is needed.

Do not remove the inner foam pads which are adhered to the plastic walker boot.

If any problems occur with the use of this product, immediately contact your medical professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Relative immobilisation of the ankle and foot at right angles.

Cushioning of the heel and plantar sole with high-density foam.

Padding of the leg and foot with the possibility of local reinforcement.

Curved, non-slip sole.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

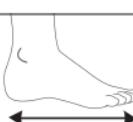
- 1) Open the three hook-and-loop fastener straps and take out the removable liner.

- 2) Lace the foot and the calf in the removable lining.

If necessary, pads can be placed inside the removable lining to fill any spaces. It is also advisable to place a pad on the top of the foot before fastening the lining for more comfort.

MEASUREMENT AND SIZE CHART

The shoe size is to choose in accordance with the length of your foot.

Size	0	1	2	3	4
Shoe sizes (UK)	2-3.5	3.5-6	6-9.5	9.5-11	11-13

3) Fasten the lining by tightening moderately with the help of the Velcro.

4) Place the foot in the boot, pulling the vertical flaps apart slightly.

• The heel should be right up against the back shell of the boot. The flaps should be centred on the axis of the leg.

• The foot / ankle and ankle / calf axes should be perpendicular.

5) Remove the protective tabs from inside the flaps to fasten the removable lining to the boot.

6) Fasten the straps again from the bottom upwards to obtain firm but comfortable support.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store away from sources of excessive heat.

Hand wash in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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