COLLIERS CERVICAUX C1 & C2

CERVICAL COLLARS C1 & C2

6625/6626/6627/6628/6629/6630/6631/6632/6642
6643/6644/6645

6633/6634/6635/6636/6637/6638/6639/6640/6641

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EN | Instructions for Use

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SV | Bruksanvisning

NO | Bruksanvisning

DA | Brugsanvisning

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FR COLLIER CERVICAL SOUPLE C1

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EN C1 SOFT CERVICAL COLLAR

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

FEATURES AND BENEFITS

The devices are soft cervical collars with removable sleeve and are fastened with a hook and loop system.

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Per minor neck injuries or conditions that require motion restriction of the cervical spine such as:

- Neck extension injuries (whiplash).
- Cervical spondylosis / Cervical Osteoarthritis (OA).
- Rheumatoid arthritis (RA) of cervical spine.
- Herniated cervical disc.

DE SET UP AND OPERATION

A trial is mandatory for a correct adaptation and a good understanding of the installation.

The first application is performed by a qualified fitter, and later applications by the user or a lay person following the same protocol.

Choose the correct height and circumference of the neck to obtain a head position with a horizontal look straight ahead.

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- Neck extension injuries (whiplash).
- Cervical spondylosis / Cervical Osteoarthritis.
- Rheumatoid arthritis (RA) of cervical spine.
- Herniated cervical disc.

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WARNING • PRECAUTION OF USE

The orthosis must not be used in direct contact with a damaged skin.

Do not wear for extended periods without medical follow-up.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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