QUADRIGIB® 3D

6681

FR

QUADRIGIB® 3D

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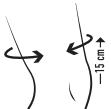
EN

QUADRIGIB® 3D

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS:

For thigh conditions that may benefit from compression, such as:

- Muscle tears in the acute phase or during rehabilitation
- Muscle pulls
- Muscle strains

PERFORMANCE

- The device provides class 2 compression.

CONTRAINDICATION

Must not be used by individuals from whom compression is contraindicated

WARNING • PRECAUTION OF USE

- The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure
- Do not wear for extended periods without medical follow-up.
- The orthosis must not be used in direct contact with damaged skin.
- If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.
- The orthosis is a single patient use device. Do not use for multiple patients
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES AND BENEFITS :

- Latex free

SET UP AND OPERATION:

- In addition to taking measurements, testing is required for a proper fitting, and for the patient to learn the proper placement.
- Thread the device by the foot, wind it up along the leg and position it around the thigh.

CARE INSTRUCTION AND MAINTENANCE

Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; Do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean. The device and packaging must be disposed of in accordance with respective local or national environmental regulations

**MEASUREMENT AND SIZE CHART**

Circumference of thigh in cm, 15cm above kneecap

1	2	3
43-51	51-59	59-67

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