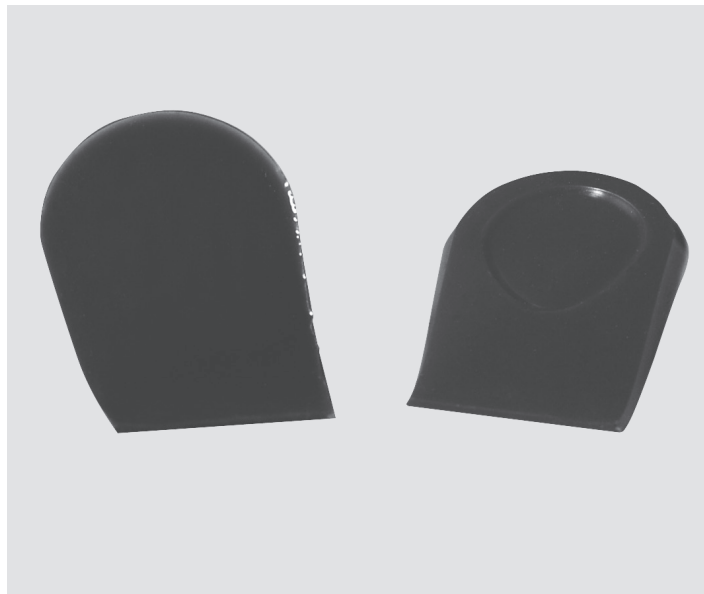
GIBAUD

# TALONNETTES HEEL LIFTS

(6369)

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**EN** | Instructions for Use

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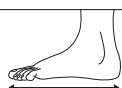
| 1     | 2     | 3     |
|-------|-------|-------|
| 34-38 | 39-42 | 43-48 |

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

#### INDICATIONS

For conditions that may benefit from use of heel lifts/heel wedges, such as:

- Achilles tendinopathy.
- Heel pain.
- Calcaneal spur.

#### PERFORMANCE

The device provides increased plantar flexion.

The device provides reduction in Achilles tendon strain.

The device provides pain relief.

#### CONTRAINDICATION

None known.

#### WARNINGS • PRECAUTIONS OF USE

A heel lift must be worn on both sides.

The use of heel lifts should be stopped gradually after symptoms have disappeared.

A fitting session is necessary for correct, compliant adjustment and for the patient to learn the correct positioning.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### FEATURES OF THE DEVICE

Heel lift, made of flexible silicone.

Central depression for better heel placement.

8 mm height at maximum, height reducing towards the front.

#### SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

A heel lift must be put in each shoe.

Position the heel riser in the posterior part of the footwear.

The depression must be positioned upwards.

#### CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the device away from sources of excessive heat.

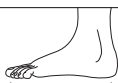
Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

#### MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the shoe size (UK).

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GIBAUD S.A.S.U