GIBAUD

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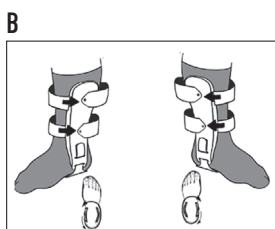
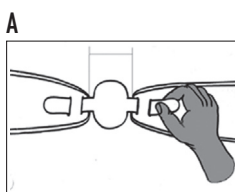
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For conditions that may benefit from ankle inversion/eversion restriction, such as:

- Acute mild to severe ankle sprains.
- Chronic ankle instability.
- Post-cast and post-walker support.
- Ankle injury rehabilitation and/or post-surgical use.

PERFORMANCE

The device provides inversion and eversion restriction to the ankle.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The device should not be used in cases where the injury requires absolute immobilization or surgical procedure.

It's forbidden to drive a vehicle whilst wearing the brace. Inspect the device carefully before applying it.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

When adjusting strap pay attention to have the tightness firm but comfortable.

Use footwear when walking outdoors or during sportive activities.

Avoid walking with the splint without footwear on slippery surfaces, particularly if the surface is wet (bathroom).

If necessary, inflate air bladders for additional compression and support. Pay attention to not overinflate.

If any problems occur with the use of this product, immediately contact your medical professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE Adaptable anatomical shape, using inflatable air cushions.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

- 1 - Adjust the distance near the heel **(A)** in such a way that the lateral struts are well-adjusted at each side of the foot. Always wear a sock under the orthosis.
- 2 - Pre-inflate the cushions. In order to do this, remove the valve located above the cushion and insert the tube that is provided; start inflating and then tighten the valve again before refolding and then glue it back against the shell.
- 3 - Set the shells of each side of the foot **(B)** so as to cover the malleolus. The rivets must be at the inner side of the leg. Straps must be tied from back to front.
- 4 - Tighten the lower strap well **(C)**.
- 5 - Close the upper strap by starting from back. The strapping must be firm but comfortable **(D)**.
- 6 - Slip the foot with the orthosis in a spacious and stable shoe: The shoe must be adjustable with laces or self-fastening tape. Avoid using these with ballerinas or pumps.
- 7 - Adjust the shoe straps.

MEASUREMENT AND SIZE CHART

The shoe size is to choose in accordance with the length of your foot.

	Size	1
	Shoe size (UK)	2 - 12

8 - Lateral cushions can be inflated, if more compression or support is required. In order to do this, remove the valve located above the cushion and insert the tube that is provided; start inflating and then tighten the valve again before refolding and then glue it back against the shell.

9 - It may loosen after it has been used for some time; in this case, tighten the straps again.

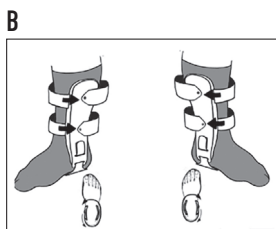
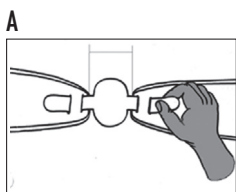
10 - If there is a local discomfort and/or tingles, open the straps and reposition the shells.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Wipe the plastic parts with a clean wet cloth.

Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a heat source (ex: radiator), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U