GIBAUD

**ATTELLE AÉRÉE  
0°/20° STANDARD**

**STANDARD LIGHT  
0°/20° SPLINT**

6911

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**EN** | Instructions for Use

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**SL** | Návod na použitie

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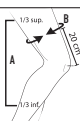
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

### INDICATIONS

For conditions that may benefit from knee immobilization, such as:

- Traumatic knee injuries.
- Emergency immobilization.
- Pre-/post-op knee immobilization.

### PERFORMANCE

The device provides motion restriction of the knee joint or relative immobilization.

### CONTRAINDICATIONS

None known.

### WARNINGS • PRECAUTIONS OF USE

The splint should not be used when the patient's condition requires absolute immobilization or a surgical procedure.

Do not wear for extended periods without medical follow-up.

The splint must not be used in direct contact with a damaged skin.

The splint is a single patient use device. Do not use for multiple patients.

When adjusting the straps, tighten them so that the hold is firm but comfortable.

If any problems occur while using the splint, e.g. pain or appearance of local signs, remove the device and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### FEATURES OF THE DEVICE

Pairs of removable lateral metal stays (0° or 20° flexion).

2 dorsal stays (1 for size T0).

1 pre-positioning strap.

4 semicircular straps.

### SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

The device can be worn on or under trousers.

1-Opening the splint:

Release the straps, including the pre-positioning strap inside the device, and pull the the hook and loop fastener part over the straps.

2- Adjusting the stays:

The product comes with 0° lateral and dorsal stays.

Two lateral stays with 20° flexion are supplied with the device and, depending on the prescription of the healthcare professional, can be used instead of the 0° lateral stays.

Where applicable, please refer to the illustrative drawings for the replacement of 0° lateral stays with 20° stays. During testing the fitter must shape the dorsal stays of the brace at 20° to adjust it to your form.

3- Securing the straps:

Attach the pre-positioning strap just above the kneecap. Tighten in such a way that the hold is firm but comfortable.

Attach and tighten, without compressing, the two straps near the kneecap (lower followed by upper) and then the two straps further from the kneecap (upper followed by lower). Tighten the straps in such a way that the hold is firm but comfortable.

### CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store the splint away from excessive heat. After removing the stays from the splint (see illustrative drawing), hand wash the splint in warm soapy water (maximum 40°C), rinse thoroughly; wring out gently; do not bleach; dry flat away from excessive heat (e.g. radiator); do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with local or national environmental regulations.

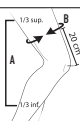
### MEASUREMENT AND SIZE CHART

The size of the orthosis is to be chosen according to :

- the length from the upper 1/3 of the thigh to the lower 1/3 of the calf of the leg to be immobilized (in cm). (B)

- the circumference of the thigh at 20 cm from the kneecap (in cm). (A)

	T0	T1	T2	T3	T4
A	30	50	50	60	60
B	30-40	45-55	55-65	45-55	55-65

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GIBAUD S.A.S.U