G I B A U D

CEINTURE LOMBAIRE LUMBAR BELT

6727

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EN | Instructions for Use

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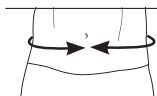
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

Lower back conditions that may benefit from gross motion restriction, such as:

- Muscular low back pain.
- Sciatica.
- Lumbosacral strains.
- Chronic low back pain.

PERFORMANCE

The device provides lumbar spine gross motion restriction.

The device reduces stress on the low back in flexion and extension.

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS • PRECAUTIONS OF USE

This product contains Naturel Rubber Latex which may cause allergic reactions.

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

Do not wear for extended periods without medical follow-up.

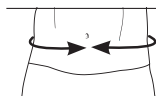
The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Gradual withdrawal possibly combined with functional physiotherapy.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the waist circumference, measured 2 cm above the iliac crests (in cm).

| 1 | 2 | 3 | 4 | 5 |
|-------|-------|--------|---------|---------|
| 70-80 | 80-90 | 90-100 | 100-115 | 115-130 |

FEATURES OF THE DEVICE

The device is a lumbar stretchable belt featuring four V-shaped posterior stays.

Ergonomic finger loops facilitate installation.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During testing, the fitter must shape the V-shaped stays of the brace to adjust

it to your morphology.

The belt is worn under the clothes or over the undergarments.

- 1) Position the bottom of the belt on the top of gluteal cleft.
- 2) Centre the dorsal rib on either side of the spinal column. The Gibaud logo must be on the right.
- 3) Tighten the belt based on the intensity of the pain and type of activity, to obtain a tighten firm but comfortable.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the belt away from direct heat sources (radiators, dryers...).

30°C mild fine wash (Hand wash); rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U