GIBAUD

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THERMAL SHOULDER SLEEVE

4298

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EN | Instructions for Use

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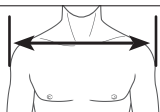
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For conditions that may benefit from the adiathermic properties of the wool, such as:

- Rheumatism.

PERFORMANCE

Thermal insulation and breathability.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up.

The shoulder sleeve must not be used in direct contact with a damaged skin. The shoulder sleeve is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the shoulder sleeve, e.g. pain or appearance of local signs, remove the shoulder sleeve and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

When fitting, put on one arm, put the shoulder strap on your back and put on the other arm.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the shoulder sleeve away from sources of excessive heat.

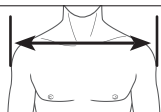
Hand wash in warm soapy water (maximum 30°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the distance between the two shoulders (in cm).

S	M	L
35-45	45-55	55-65

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GIBAUD S.A.S.U