GIBAUD

GENUGIB® ROM

6902

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EN | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For knee conditions that may benefit from increased stability around the knee, such as:

- Mild to moderate ACL or PCL, MCL and LCL instabilities, sprains and strains.

PERFORMANCE

The device shall provide increased stability around the knee.

The device provides limitation in flexion and extension of the knee joint.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization by cast or resin or surgical procedure.

Do not wear for extended periods without medical follow-up.

When adjusting the straps, tighten them so that it is firm but comfortable.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

2 lateral polycentric hinges.

Wedges to adjust amplitude of flexion and extension.

6 adjustment straps (4 front and 2 back).

Fully opens at the front.

2 knee pads (half-rings).

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

The device can be worn over or under trousers.

When fitting the device, the professional should adapt the hinges to your morphology, if necessary.

1) For adjusting the range of motion:

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the knee (in slightly bent position), measured at the center of the patella (in cm).

1	2	3	4	5	6
31-34	34-37	37-40	40-43	43-46	46-50

It must imperatively be carried out by the healthcare professional before fitting the orthosis.

- 1 - Unscrew and remove the wedge to adjust the amplitude on the chosen side (in accordance with the restriction (extension/flexion) recommended by the prescriber (see drawings 1, 2, 3, 4)).
- 2 - Fit the new wedge to adjust the amplitude (flexion angles: 0°, 45°, 60°, 75°, 90°; extension angles: 0°, 10°, 20°, 30°, 40°) in accordance with the amplitude recommended by the prescriber (see drawing 5).
- 3 - Screw the wedge back on (see drawing 6).

2) For the setting of the orthosis:

The GIBAUD logo should be on the front at the top of the device when it is correctly fitted.

- 1 - Loosen the two rear straps and fold the hook and loop part onto the straps.
- 2 - Detach the four front straps and fold the hook and loop part onto the straps.
- 3 - Fully open up the orthosis.
- 4 - Make sure to place the two blue patellar half-rings on either side of the patella and close the hook-and-loop openings / fastenings, starting with the lower one (see illustrative diagram).
- 5 - The closing order of the straps is indicated by the numbering (from 1 to 6) dot(s) in the circles (see illustrative

diagram).

- 6 - Tighten the straps for a firm but comfortable fit.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep brace away from direct heat sources.

After hinges have been removed from the orthosis (see the illustrative diagram with the description of the steps), hand wash the orthosis in warm soapy water (maximum 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, away from direct heat sources (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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EN

- ① Remove the product. Leave the straps loose. The hinges are visible on the sides of the orthosis.
- ② Pull to detach the center.
- ③ Tilt the hinge to the maximum, and pull to extract it.
- ④ To put the hinge back in place, tilt the hinge to the maximum, fold the orthosis completely so that the two pockets are facing the metal bars and slide them inside, reattaching the center.

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EN: Illustrations for step 1 - Limitation of extension

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GIBAUD S.A.S.U