GIBAUD

# MANUGIB<sup>®</sup> 3D

6700 / 6701

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**EN** | Instructions for Use

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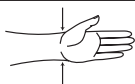
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

## INDICATIONS

For conditions that may benefit from compression over the wrist, such as:

- Wrist sprain and strains.
- Overuse and pain conditions.
- Wrist tendinopathy.
- Rheumatic Arthritis (RA) and other chronic pain conditions.

## PERFORMANCE

The product shall restrict motion of the wrist.

Pain relief for chronic wrist pain.

The device provides compression over the wrist.

## CONTRAINDICATION

Must not be used by individuals from whom compression is contraindicated.

## WARNINGS • PRECAUTIONS OF USE

When adjusting the strap, tighten it so that it is firm but comfortable

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with damaged skin.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The orthosis is a single patient use device. Do not use for multiple patients

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

- Latex free.

## SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During the first fitting, the fitter must shape the splint to the palm according to the patient's body and activity.

1 - Remove the strap.

2 - Slip the device on like a glove, taking making sure that the end of the splint is positioned in the palm of the hand, and runs along the forearm.

3 - Place the strap's loop over the hook-and-loop strip on top of the wrist.

4 - Make sure that the bone in the wrist (the ulna styloid process) is positioned in the hold in the strap intended for this purpose.

5 - Slide the strap through the loop.

6 - Tighten to obtain a firm but comfortable hold.

## CARE INSTRUCTION AND MAINTENANCE

Remove splint from pouch before cleaning or repairs.

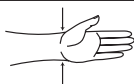
Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

To determine the size, measure the wrist circumference and compare to the size table below. Choose the correct product for left or right wrist.

1D-1G	2D-2G	3D-3G	4D-4G	5D-5G	6D-6G
14-15	15-16	16-17	17-18	18-19	19-20

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GIBAUD S.A.S.U