G I B A U D

MANUGIB[®] RHIZACTIVE

6704

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EN | Instructions for Use

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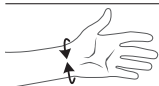
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATION

Thumb Carpometacarpal (CMC) Osteoarthritis (OA).

PERFORMANCE

The brace provides stabilization of the thumb.

The brace provides functional improvements for CMC OA.

The brace provides pain relief for CMC OA.

CONTRE-INDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

When adjusting the strap, tighten it so that it is firm but comfortable.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

The two thumb adjustment tabs provide firm but comfortable support.

Fleece lining.

Adjustment strap.

The pad under the tightening loop prevents compression of this loop.

Ambidextrous orthosis.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During fitting, the applicator must shape, if necessary, the splints of the orthosis to adapt them to your morphology.

1 - Open the wrist tab; loosen the thumb tabs without opening them.

2 - Pull the orthotic device over your hand like a glove, sliding your thumb through the thumb straps.

3 - Close up the orthotic device using the thumb tab, so that the hold is firm but comfortable.

4 - Adjust the thumb straps in such a way that the hold is firm but comfortable. Next, position the straps along the thumb column. Make sure the straps are parallel to the central seam, without overlapping it.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the orthosis away from sources of excessive heat.

Hand wash in warm soapy water (maximum 40°C), rinse thoroughly.

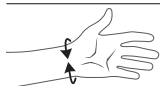
Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the wrist, below the styloid (in cm).

1	2
13,5-17	17-20

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GIBAUD S.A.S.U