G I B A U D

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS

Situations requiring inversion/eversion restriction. These may include:

- Acute mild to severe ankle sprains.
- Chronic ankle instability.
- Post-cast and post walker support.
- Ankle injury rehabilitation and/or post-surgical use.

PERFORMANCE

The device provides inversion and eversion restriction to the ankle.

The device improves proprioception.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

It's forbidden to drive a vehicle whilst wearing the brace.

It is advisable to wear the product with a sock.

When adjusting the straps pay attention to have the tightness firm but comfortable.

Use footwear when walking (outdoors or during sportive activities).

Avoid walking with the splint without footwear on slippery surfaces, particularly if the surface is wet (bathroom).

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Rigid shells.

Tightening by sliding buckles.

Resilient foam.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

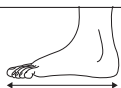
1) Adjust the width at the ankle so that the side supports are correctly fitted on each side of the foot.

2) Always wear a sock under the orthosis. Fit the protective shells over the malleoli on each side of the foot. The buckle must be placed on the outside of the leg.

3) Firmly apply the lower strap to have the tightness firm but comfortable.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the shoe size (UK, single size).

1

2.5-11

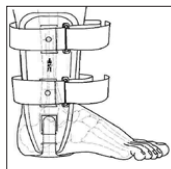
4) Slip the foot fitted with the orthosis into a spacious and stable shoe. The shoe must be closed with laces or hook and loop system straps. Never use loafers, ballerina or casual shoes.

5) Adjust the tightness of the shoe.

6) Apply the upper strap from the back. All strapping must be firm and comfortable.

7) Retighten the straps if they loosen after a wearing for a certain time.

8) If local difficulties and/or tingling occur, loosen the straps and reposition the shells.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the device away from sources of excessive heat.

Wipe plastic parts with a clean moist cloth. The removable side protection may be machine washed (fragile program, low temperature).

Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U