G I B A U D

# POIGNET DE FORCE

## STRONG WRIST

6420

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**EN** | Instructions for Use

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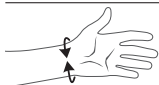
| 1     | 2     | 3     |
|-------|-------|-------|
| 14-16 | 17-19 | 20-22 |

**If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.**

## INDICATIONS

For conditions that may benefit from support and stabilization to the wrist, such as:

- Activities with high level of strain and/or vibration.
- Mild sprains.
- Strains.
- Complications after trauma injuries.

## PERFORMANCE

The brace shall provide support and stability to the wrist.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

This product contains Natural Rubber Latex which may cause allergic reactions.

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

When adjusting the strap, tighten it so that it is firm but comfortable.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to

the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

Adjustable tightening.

Closing system with pulley effect.

Foam liner.

## SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1) Thread the bracelet on the wrist, the buckle must always be positioned on the right.

2) Tighten the strap to obtain a firm but comfortable hold.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the device away from sources of excessive heat.

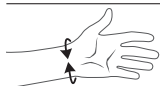
Hand wash in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the wrist below the styloid (in cm).

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GIBAUD S.A.S.U