# GIBAUD

## SOIN DU PIED FOOT CARE

SOIN DE  
L'HALLUX  
VALGUS

HALLUX  
VALGUS CARE

CAMÉLÉA BOTTINE HV

CAMÉLÉA SANDALE HV

CAMÉLÉA HV

CAMÉLÉA NU-PIEDS HV

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DAILY FOOT  
CARE

CERVIA FLEX

BISENTINA

TROPÉA

RIVIERA

FORLI

FEMME/WOMAN

HOMME/MAN

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**If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.**

## INDICATIONS

For foot conditions that may benefit from volume accommodation, such as:

- Foot deformation.
- Atypic foot morphology.

## PERFORMANCE

Large forefoot part of the shoe for accommodation to high volume (perimeter of the metatarsis).

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

This device may contain Natural Rubber Latex. Latex may cause allergic reactions. The presence of latex is indicated by the symbol  on the product composition label.

Do not wear for extended periods without medical follow-up.

The shoes must not be used in direct contact with a damaged skin.

The shoes are single patient use devices. Do not use for multiple patients.

If any problems occur with the use of the shoes, e.g. pain or appearance of local signs, remove the shoes and contact your healthcare professional.

The user and/ or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

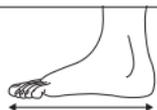
To maintain your shoes, start by cleaning with a soft shoe brush or a damp cloth. This is to remove the dust of your shoes. Then, it is advisable to spray a waterproofing product to nourish and protect from moisture. We recommend that you use soft products purchased from a specialized distributor. These products are to be applied regularly in small quantities. Never use caustic or corrosive cleaning products. Never allow wet shoes to dry near a heat source; do not tumble dry, do not iron.

Cleaning varies according to the frequency of use. If the use is daily, it is advisable to maintain your shoes at least once a month.

Dispose of the device and its packaging in accordance with local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

Size of the shoes to choose according to the foot size (UK).

Woman	Man
2.5*-8	6-13*

\* For certain models only.

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

## INDICATIONS

For foot conditions that may benefit from volume accommodation, such as:

- Foot deformation like Hallux Valgus.

## PERFORMANCE

Adaptable zone on the forefoot of the shoe to accommodate the volume of the forefoot (metatarsian internal part).

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

This device may contain Natural Rubber Latex. Latex may cause allergic reactions. The presence of latex is indicated by the symbol  on the product composition label.

Do not wear for extended periods without medical follow-up.

The shoes must not be used in direct contact with a damaged skin.

The shoes are single patient use devices. Do not use for multiple patients.

If any problems occur with the use of the shoes, e.g. pain or appearance of local signs, remove the shoes and contact your healthcare professional.

The user and/ or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

To maintain your shoes, start by cleaning with a soft shoe brush or a damp cloth. This is to remove the dust of your shoes. Then, it is advisable to spray a waterproofing product to nourish and protect from moisture. We recommend that you use soft products purchased from a specialized distributor. These products are to be applied regularly in small quantities. Never use caustic or corrosive cleaning products. Never allow wet shoes to dry near a heat source; do not tumble dry, do not iron.

Cleaning varies according to the frequency of use. If the use is daily, it is advisable to maintain your shoes at least once a month.

Dispose of the device and its packaging in accordance with local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

Size of the shoes to choose according to the foot size (UK).

Woman	Man
2.5*-8	6-13*

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GIBAUD S.A.S.U