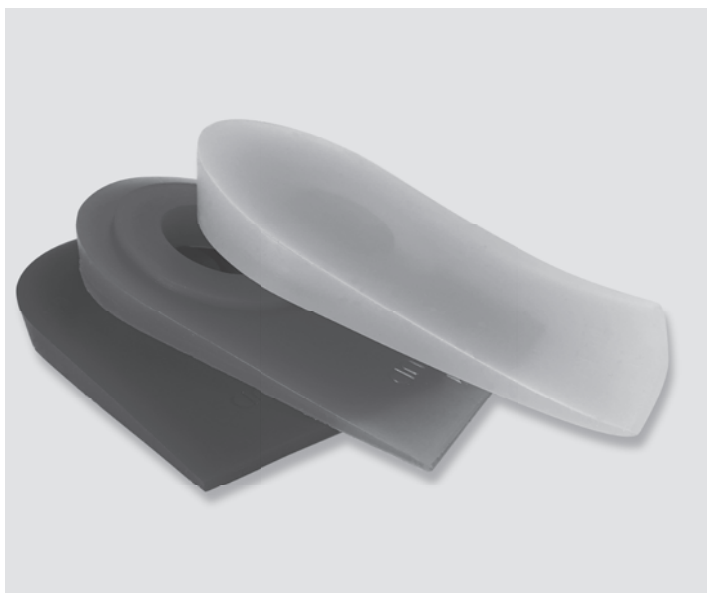
G I B A U D

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HEEL LIFTS EVOLUTION

(6392)

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EN | Instructions for Use

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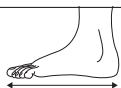
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS

For conditions that may benefit from use of heel lifts, such as:

- Rehabilitation of Achilles tendon post-surgical repair;
- Rehabilitation of Achilles tendon during non-surgical treatment.

PERFORMANCE

The device provides increased plantar flexion.

The device can be used in relation to rehabilitation of Achilles tendon post-surgical repair.

The device can be used in relation to rehabilitation of Achilles tendon during non-surgical treatment.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

Under medical guidance, progressively remove the layers, starting with the layer in contact with the shoe (the darkest colour). The lightest heel lift should be removed last (after the symptomatology has disappeared).

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report

any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

27 mm heel lift, composed of 3 separable 9 mm layers.

Central depression for better heel placement.

SET UP AND OPERATION

The first trial should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

-Insert the heel lift in a suitable shoe: the darkest colour is down and the lightest colour is on the top.

-Under medical guidance, progressively remove the layers, starting with the layer in contact with the shoe (the darkest colour).

-Important note: never remove the lightest heel lift.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

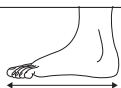
Keep the device away from sources of excessive heat.

Hand wash in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean. The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the shoe size (UK).

1	2	3
2-5	5-8	9-13

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GIBAUD S.A.S.U