GIBAUD

LOMBOGIB® MATERNITY

6652/6653

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EN | Instructions for Use

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SV | Bruksanvisning

NO | Bruksanvisning

DA | Brugsanvisning

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Taille 1

80-135 cm

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

Conditions that require sacroiliac and supra pubic support, such as:

- Low back pain related to pregnancy.

PERFORMANCE

The device provides support.

The device reduces low back pain.

CONTRAINDICATION

The device is safe to use as intended.

WARNINGS • PRECAUTIONS OF USE

The orthosis must not be used in direct contact with a damaged skin

This product contains Naturel Rubber Latex which may cause allergic reactions.

The orthosis is a single patient use device. Do not use for multiple patients.

The belt must never compress the abdomen.

Do not wear for extended periods without medical follow-up

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Lombogib Maternity is an extensible brace featuring V-shaped posterior stays.

The brace consists of two independent parts, connected by a hook and loop system.

A soft back pad can be added/removed. The height of the belt is 26 cm.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During testing the fitter must shape the V-shaped stays of the brace to adjust it to your form.

1) Measure the waist measurement at the level of the navel in order to choose the appropriate abdominal support:

- <100 cm: small abdominal support;
- > 100 cm: large abdominal support.

2) Hook one end of the abdominal support to the posterior part, taking care that the whole hook-and-loop system is covered.

3) Position the posterior part of the belt to have the stays on either side of the spine.

4) Position the abdominal support under the abdomen, so that the morpho-adaptable V-shaped part is centered under the navel.

5) Close the belt by hooking the second end of the abdominal support to the posterior part.

6) Adjust the abdominal support symmetrically, to obtain a compression-free tightening.

7) Adjust the posture-correcting straps to obtain optimized support without compressing.

Ability to adjust the lumbar padding in height according to the camber by positioning it symmetrically to the spine

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

Store the brace away from sources of extreme heat (including radiators and tumble dryers).

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

The device has a unique size. For a waist circumference, measured 2 cm above the iliac crests (in cm) of 80-135 cm.

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GIBAUD S.A.S.U