GIBAUD

SOIN DES PIEDS DEFORMÉS DEFORMED FEET CARE

TEXTILE

 OLYMPIE  RHODES  LEVITHA

 SPARTE  SANTORIN  THILIA

 ALEXANDRIE  KÉA

 CORINTHE  IKARIA

 DELPHES  ATHÈNES

 TIVOLI  OLIVER

CUIR/AUTRES MATÉRIAUX LEATHER/OTHER MATERIALS

 MILO  CORFOU  PISA

 ANAFI  OLBIA  FOGGIA

 ÉGINE  MATERA  PETILLA

 PADOU  TRANI  BIELLA

 THERA  SERIFOS  SPEZIA

 GELA  MARESCA  EVIA

 BARI  PALERME  BERGAME

 FEMME / WOMAN  HOMME / MAN  MIXTE / MIXED

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS

For foot conditions that may benefit from volume accommodation, such as:

- Inflammation or edema with (risk of) skin disorders.
- Post-surgery.
- Lymphoedema.
- Wound dressings.
- Foot deformation like Hallux Valgus*, rheumatologic foot.

*Note: Hallux Valgus only applicable for Milo, Anafi, Cythère, EGINE, Lipsi, Petilla, Spezia and Trani.

PERFORMANCE

Large forefoot part of the shoe for accommodation to high volume (perimeter of the metatarsis).

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up.

This device may contain Natural Rubber Latex. Latex may cause allergic reactions. The presence of latex is indicated by the symbol  on the product composition label.

The shoes must not be used in direct contact with a damaged skin.

The shoes are single patient use devices. Do not use for multiple patients.

If any problems occur with the use of the shoes, e.g. pain or appearance of local signs, remove the shoes and contact your healthcare professional.

The user and/ or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

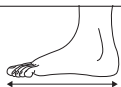
SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend. Adjust shoe closure systems to the patient's morphology to obtain good foot hold.

For the THERA model, during the first donning, the tightening must be done only with the laces and not with the zipper (zip) to adapt the volume of the shoe. Subsequently, removal and donning can be done using the slider (zip).

MEASUREMENT AND SIZE CHART

Size of the shoes to choose according to the foot size (UK).

Woman	Man
2.5*-8	6-13*

* For certain models only.

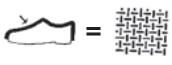
CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

To maintain your shoes, we recommend that you use soft products purchased from a specialized distributor. These products are to be applied regularly in small quantities. If your shoes are wet, we recommend that you never dry them near a heat source; do not tumble dry, do not iron.

For maintenance, refer to the sticker located inside the shoe. Standard symbols tell you the composition; you will find the recommendations in the table below.

Cleaning varies according to the frequency of use. If the use is daily, it is advisable to maintain your shoes at least once a month.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LEATHER / OTHER MATERIALS	TEXTILE
 <p>Shoe upper = Leather Coated Leather and/or Other materials</p>	 <p>Shoe upper = Textile</p>
<ul style="list-style-type: none"> - Start by cleaning with a soft shoe brush or a damp cloth to remove the dust. - It is advisable to spray a waterproofing agent on all leather or synthetic shoes to nourish and protect from moisture. - Never use caustic or corrosive cleaning products. 	<ul style="list-style-type: none"> - Machine washable, never exceed a temperature of 30°C, short program. - Add a cloth soaked in liquid detergent, without excess, to the drum.

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GIBAUD S.A.S.U