GIBAUD

LOMBOGIB® PROFORCE

6746/6747

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EN | Instructions for Use

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If you have any uncertainty regarding the applications and use of this medical device, please ask your doctor and/or orthopaedist-orthotist, pharmacist or orthopaedic consultant for advice.

INDICATIONS

For lower back conditions that may benefit from gross motion restriction, such as:

- Muscular low back pain.
- Sciatica.
- Lumbosacral strains.
- Chronic low back pain.

PERFORMANCE

The device provides lumbar spine gross motion restriction.

The device reduces stress on the low back in flexion and extension.

The device provides pain relief.

The device increases proprioception.

The device improves postural control/stability.

CONTRAINDICATION

The device must not be used by individuals for whom compression is contraindicated.

WARNING • PRECAUTIONS FOR USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

This product contains Natural Rubber Latex (in the elasticated fabrics) which may cause allergic reactions.

The orthosis must not be used in direct contact with a damaged skin.

Do not wear for extended periods without

medical follow-up.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Gradual withdrawal possibly combined with functional physiotherapy.

FEATURES OF THE DEVICE

The device is an elastic lumbar support belt comprising:

- **(1A)** Four conformable dorsal ribs.
- **(1B)** Two front hook-and-loop fastener areas fitted with finger loops.
- **(1C)** Dual tightening, consisting of four straps, tensioned by means of a system of two force reduction pull tabs.

FITTING / REMOVAL

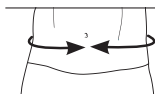
A fitting is required to ensure a good fit and a good understanding of how to put the device on. A healthcare professional should fit the device on the patient for the first time; thereafter, it should be put on the patient in the same way by the user themselves or a family member/friend. When fitting the orthosis, the professional should adapt the orthosis's dorsal ribs to fit the patient's body shape.

Before fitting:

- **(2A)** Open the belt, lay it flat and detach the pull tabs from the hook-and-loop fasteners.

MEASUREMENT AND SIZE CHART

Choose the belt size based on the circumference of the waist, measured 2 cm above the iliac crest. The device is available in two heights: 21 cm (6747) and 26 cm (6746).

0	1	2	3	4	5
66-80	80-90	90-100	100-110	110-125	125-140

- (2B) Depending on the patient's needs, the second tightening can be height-adjusted.

Fitting:

- (2C) Pick up the belt so that the Gibaud logo is on the right.
- (2C) Position the bottom of the belt over the top of the intergluteal cleft.
- (2C) Centre the dorsal ribs on either side of the spinal column.
- (2D) Use the finger loops to tension the belt and secure it, to achieve a firm but comfortable fit.
- (2E) Use the pull tabs to tension the second tightening, to achieve a firm but comfortable fit.
- (2F) Hook the pull tabs onto the hook-and-loop fasteners on the front of the belt.

The tightness of the belt can be adjusted according to pain and activity.

Removing the belt:

- Unhook the pull tabs.
- Open and remove the belt.

STORAGE

Store the orthosis away from light, moisture and extreme heat (including radiators and tumble dryers).

MAINTENANCE AND CARE

Preparation for washing:

Close the belt and reposition the pull tabs on the hook-and-loop fasteners.

Washing:

Gently wash in warm soapy water at 30°C (hand wash or machine wash on a delicates cycle); rinse thoroughly; wring out gently; do not bleach; lay flat to dry, do not tumble dry; do not iron; do not dry clean.

DISPOSAL

The device and packaging must be disposed of in accordance with the relevant local or national environmental regulations.

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GIBAUD S.A.S.U